**N E O M E D**

## **Instructiuni de folosire**

---

### **LAMPA GERMICIDA**

**G A R A 2 x 30 W K –montare perete**

**G A R A 2 x 30 W K / M- pe suport mobil**

**NEOMED, s.r.o.**

**Gaštanová 2**

**066 01 Humenné**

**Tel. 057 7884222-0, Fax 057 7884222**

**Email: [neomed@neomed.sk](mailto:neomed@neomed.sk), [www.neomed.sk](http://www.neomed.sk)**

**CE** 2265

Draga client,

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# DECLARATION OF CONFORMITY

in compliance with § 23 law no. 56/ 2018 Coll. about technical requirements of products and conformity assessment and amendment to certain laws, as amended,

Manufacturer : NEOMED, s.r.o., Gaštanová 2, 066 01 Humenné

Device identification: Germicidal emitter  
Variants : GARA 30WO,GARA30WO/C, GARA 30WO/KL  
GARA 30WO/M, GARA 30WU, GARA 30WU/M,  
GARA 2x30WK, GARA 2x30WK/M

Medical device :  
Class :

I

Used standards : STN EN 60598-1:2015,STN EN 60698- 2-1, STN EN 62493,  
STN EN 61347-1, STN EN 61347-2 -3

Conformity assessment procedure according to : Modul A1

Name and code of notified  
Body (NB):

3EC International , NB 2265

Certificate Quality  
Management system:

ISO 9001: 2015, No. Q-0212/19, validity 17.08.2022

We hereby declare that the product above is safe for its intended use.

In the production process, the conformity of all products placed in the market with the technical documentation is ensured. The properties of the product meet the technical requirements that apply to this product and which are specified in the relevant government regulations, or in the other technical regulations.

All documents, including technical documentation, are kept at manufacturers'.  
This medical device is CE marked.

Place : Humenne, date 17.08.2019

Name and function of signatory empowered  
to bind the manufacturer : Ján Kľučár , director

Signature :  
Stamp :

**NEOMED, s.r.o.**

Gaštanová 2  
066 01 Humenné  
Slovakia

tel/fax: + 421 57 788 4222

e-mail: [neomed@neomed.sk](mailto:neomed@neomed.sk)

<http://www.neomed.sk>

## GUARANTEE

<b>Product</b>	<i>Germicidal emitter</i>	
<b>Type</b>	<i>GARA 2x 30 WK</i>	<b>Serial number</b>
guarantee for quality and functions of product dutiny		
months from day of sale, but max	12	months from delivering

### CONDITIONS OF GUARANTEE:

In the agreed time organisation repairs all kinds of disorders, under this conditions

- a, usage of device was under recommendation of producer
- b, device was not overloaded
- c, usage of device was not unprofessional
- d, device was placed on recommended

Device in guarantee cannot be dismantled.

### CERTIFICATE OF INSPECTION

NEOMED, s.r.o. certifies that product described above has been inspected and found conform to the applicable international standards and NEOMED standards.

Stamp of producer:

Day of sale: 5/2020

Sign of examination:

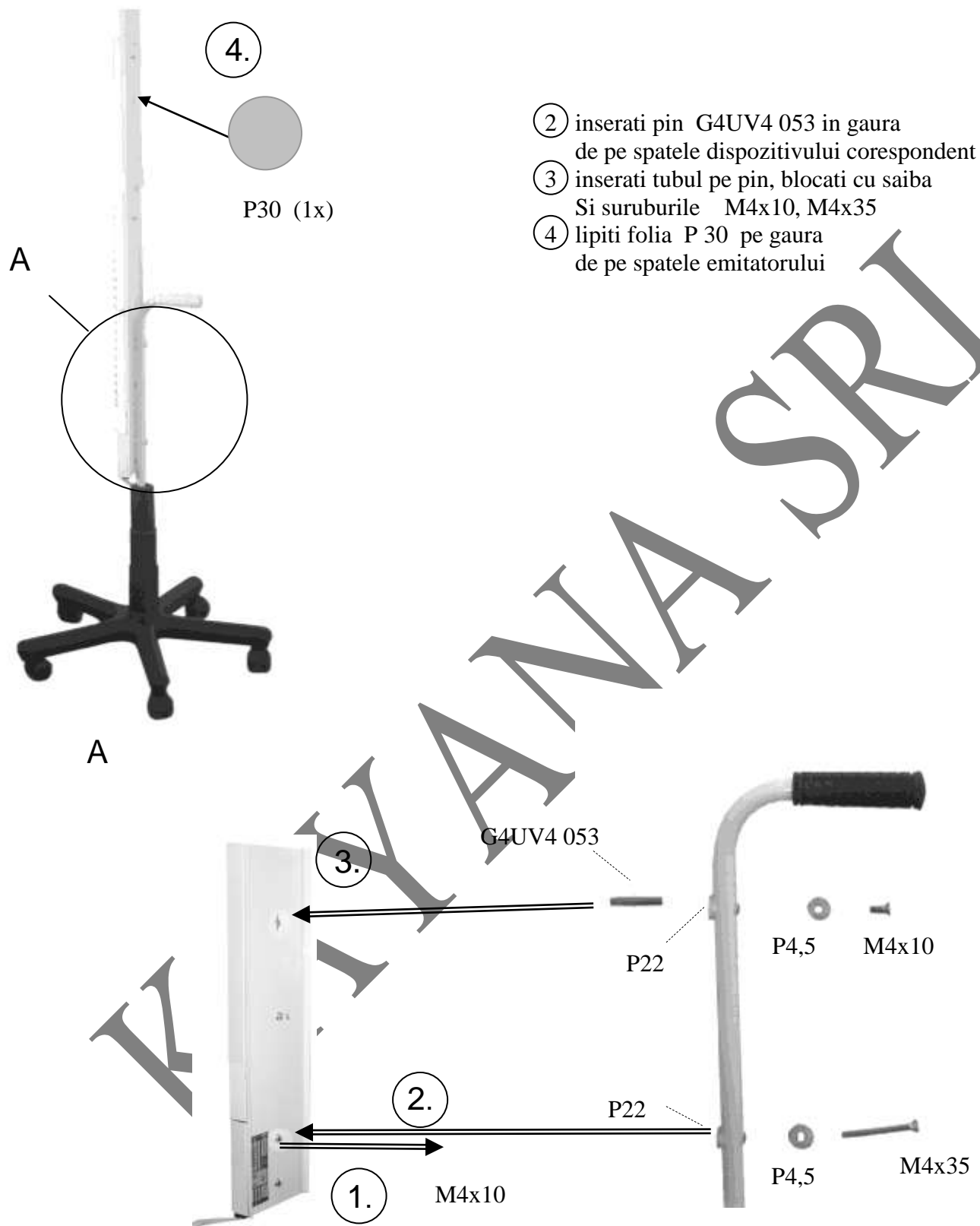
### REPORT ABOUT INSTALATTON OF DEVICE

Day of install	Stamp of install. company	Tel./Fax	Singnature

### INFORMATIONS ABOUT REPAIRS

Description of disorder	Day of repair	Stamp of service company

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